R_x ONLY


CBCII Collection Bag

STERILE R

REF 0225-028-851

ENGLISH (EN)
ESPAÑOL (ES)
DEUTSCH (DE)
FRANÇAIS (FR)
ITALIANO (IT)
NEDERLANDS (NL)
日本語 (JA)

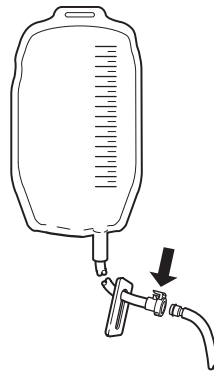EN

For Use with CBCII Quick Disconnect Kits only

The Collection Bag is used as a blood disposal bag if reinfusion has been discontinued and the CBCII is functioning solely as a wound drainage device.

Not made with natural rubber latex.

WARNINGS:

- Adherence to universal blood and body fluid precautions is highly recommended when handling any blood-related product.
 - Upon initial receipt and before use, inspect the package for damage and confirm the integrity of the sterile barrier. DO NOT use any equipment if damage is apparent or the sterile barrier has been compromised.
 - DO NOT reuse, reprocess, or repack a device that is intended for single use only.
 - A single use device may not withstand chemical, chemical vapor, or high temperature sterilization reprocessing.
 - Design features may make cleaning difficult.
 - Reuse may create a contamination risk and compromise structural integrity resulting in operational failure.
 - Critical product information may be lost during repackaging.
- Failure to comply may lead to infection or cross infection and result in patient and/or healthcare staff injury.
- ALWAYS follow the current local regulations governing biohazard waste to safely handle and dispose of surgical waste.
 - Label the bag appropriately to prevent reinfusion.

Instructions

Please read and understand these instructions carefully. Familiarization with the 3T Medical CBCII prior to use is important.

- If tubing or blood bag contains fluid, use slide clamps to squeeze tubing shut.
- Remove blood bag at quick disconnect and replace with 3T Medical Blood Disposal Bag.
- Unclamp tubing.

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3T Medical Systems, Inc.