Rap-Round®

REF	8001-061-530
REF	8001-061-535
REF	8001-061-540
REF	8001-061-550
REF	8001-061-560

Operations Manual

Bedjningsmanual

Manuel d'usage

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Introduction

This manual assists you with the operation of the 3T Medical Model 8001-061-530, 8001-061-535, and 8001-061-540 **Rap-Round®** hyper/hypothermia vest or leg wrap. Read this manual before operating this product and keep a copy on file. Set methods and procedures to educate and train your staff on the safe operation of this product.

Intended use and Contraindications
Always consult the controller operations manual before using this product.

Product description
The **Rap-Round** is a vest or leg wrap that wraps around the chest or leg. The **Click-Tite®** connectors provide the means to connect to the temperature controller.

Expected life
These **Rap-Round** products have a 30 day single patient use expected life upon first use and under normal use conditions.

Specifications			
Material	polypropylene, ethylene vinyl acetate, silicone, polyvinyl chloride (PVC)		
Model	8001-061-530	8001-061-535	8001-061-540
Size	small/medium chest 	large chest 	one size for the left or right leg - thigh circumference 
	32 to 46 in.	81 to 117 cm 46 to 54 in.	117 to 139 cm 20.5 to 28.5 cm 52 to 72 cm
Compatible controllers	Medi-Therm series temperature controller Atrix® Precision Temperature Management System		
	Note: Consult your local representative for controller availability.		
Kit	8001-061-550 8001-061-560		
Environmental conditions	Operation		Storage and transportation
Temperature			
Relative humidity			

3T Medical reserves the right to change specifications without notice.

Symbols

General warning	Caution
Refer to instruction manual/ booklet	Consult instructions for use
Temperature controller	Quantity
Do not puncture	Single patient use
Batch code	Catalogue number
Date of manufacture	Manufacturer

CAUTION
US Federal Law restricts this device to sale by or on the order of a licensed physician.

Contact information

Contact 3T Medical Customer Service or Technical Support at +1 (734)525-9580

3T Medical Systems, Inc.
236 Fernwood Avenue
Edison, NJ 08837
USA

To view your product manual online, see <https://3tmedical.com/technical-docs>

Have the Lot batch code (A) (Figure 1) available when you call Customer Service. Include the Lot batch code in all written communication.

Safety precautions

WARNING

- Always check the product for leaks, tears, cuts, holes, stains, or any other damage prior to and during use.
- Always check the patient's skin condition in areas of contact with the blanket according to hospital protocol or as directed by a physician.
- Always turn or reposition the patient over the duration of therapy according to hospital protocol.
- Always keep the area between the patient and the thermal transfer device dry.
- Application of heat or cold are most effective when you apply the body wrap directly to the skin.
- Do not allow hoses to fold, kink, or to wrap around a patient.

CAUTION

- Do not use if the package is damaged.
- Always use **Rap-Round** products with an approved 3T Medical temperature controller. Using this product with other controllers has not been tested.
- Do not reuse this product on another patient to avoid the risk of cross-contamination and infection.
- Do not use this product to position or transfer a patient.
- Always pre-fill the vest or leg wraps to lower the risk of pressure ulcers.
- Always follow hospital protocol when regulating temperature with external devices. Frequency of assessment and documentation will vary upon the individual response of the patient.
- Always consult the controller operations manual before connecting this product.
- Always complete a finger check, after applying a vest or wrap, to make sure that it is not on the patient too tight. This check is to avoid the risk of pressure ulcers.
- Do not clean this product. If soiled, dispose of product according to hospital protocol. This is a single patient use product.
- Do not use sharp objects or pins with this product.
- Do not block fluid pathways of the thermal transfer device with restraints.

Operations

Assessing the patient's skin condition

Note: Follow your facilities skin care protocol. The following instructions are recommendations. If your facility has a different protocol, follow that protocol.

- Document initial skin assessment.
- Check for any foreign items such as medical patches, IVs, sutures.
- Check for fluids such as moisture or gel.
- Make sure that the vest or leg wrap size is correct and pre-filled before applying to the patient.

Connecting and pre-filling the vest or leg wrap (Click-Tite connectors)

- Close the pinch clamps on the controller hose and vest or leg wrap (Figure 2).
- Open all clamps on the connector hose and the vest or leg wrap (Figure 2).
- Attach the connector hoses to the vest or leg wrap. Insert the male coupling of vest or leg wrap into the female coupling of hose with a twisting motion (Figure 3 and Figure 4).
- Snap the fitting into place (Figure 5 and Figure 6).
- With a light pull, make sure that you locked all the connectors.
- Open all clamps on the connector hose and the vest or leg wrap (Figure 7).
- Insert the power cord into a wall outlet.
- Press the power button to turn on the controller. See the controller instructions for use.
- Check that the vest or leg wrap fills with water.
- Note:** Iodine based disinfectants will stain the product. Staining does not affect the patient nor the use of the product.

Applying the vest or leg wrap to the patient

- Apply the pre-filled vest or leg wrap directly to the patient's skin with the logo on the outside.
- Close the vest or leg wrap by fastening the **Velcro®** closures.
- Slide a finger along both sides of the vest or leg wrap (top and bottom). Check for constriction of any soft tissue.

Notes:

- Application of heat and cold are most effective when you apply body wraps directly to the skin.
- Make sure there are no folds or creases as those may prevent the flow of water.
- Locate the cutout part of the leg wrap over the knee.
- Avoid positioning the vest wrap edges above the bony prominence or areas of sensitive skin.
- Place the hoses on the patient's left side. Do not wrap the hoses around or under the patient.
- Following the application of the vest or leg wraps, you may see the pattern of these products on the patient's skin. This pattern on the skin may redden (hyperemic). This occurrence is normal. The reddening will disappear within the same length of time that you applied the thermal transfer device. This phenomenon is often mistaken for injury, but is in fact reactive hyperemia.^{1,2}

Rechecking the patient's skin condition

Check the patient at regular intervals as directed by hospital protocol. Open the **Velcro®** closures to check the entire area of skin that is in contact with the vest or leg wrap. Note any change in the skin integrity that relates to:

- Excessive moisture - dry the skin surface by wiping away the moisture
- Color of the epidermis
- Skin texture
- Patient's skin condition is acceptable to continue therapy

Note: Check the vest or leg wrap position, readjust the **Velcro®** closures as necessary.

Rechecking the vest or leg wrap

Recheck the vest or leg wrap for the following:

- Position
- Water status
- Dry surface
- Leaks
- Cracks

Defibrillation considerations

- Open the vest to expose patient's chest.
- Remove excess moisture.

Disconnecting the vest or leg wrap from the hoses

- Power off the controller. Allow most of the water to drain back into the machine (approximately 10 minutes).
 - Close vest or leg wrap clamps (Figure 2).
 - Close the connector hoses.
 - Unsnap the fitting. Pull the male coupling out to disconnect.
 - Discard vest or leg wrap after use per facility protocol.
- Note:** Do not discard the controller connector hose after use, save for next use.

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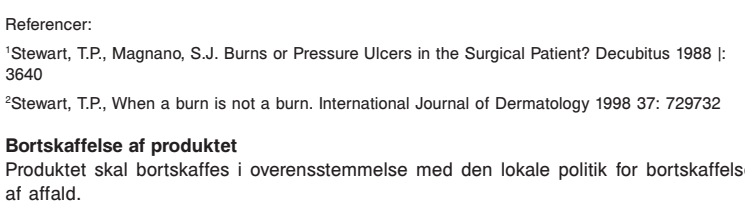
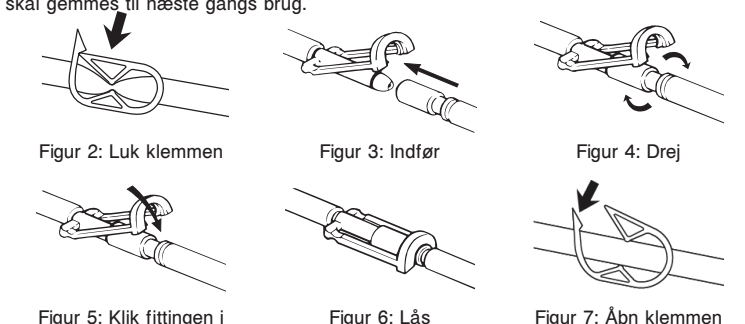
Figure 5: Snap fitting Figure 6: Lock Figure 7: Open the clamp

Figure 8: Klik fittingen i Figure 9: Lås Figure 10: Åbn klæmmen

Figure 11: Snap fitting Figure 12: Lock Figure 13: Open the clamp

Figure 14: Klik fittingen i Figure 15: Lås Figure 16: Åbn klæmmen

Figure 17: Snap fitting Figure 18: Lock Figure 19: Open the clamp

Figure 20: Klik fittingen i Figure 21: Lås Figure 22: Åbn klæmmen

Figure 23: Snap fitting Figure 24: Lock Figure 25: Open the clamp

Figure 26: Klik fittingen i Figure 27: Lås Figure 28: Åbn klæmmen

Figure 29: Snap fitting Figure 30: Lock Figure 31: Open the clamp

Figure 32: Klik fittingen i Figure 33: Lås Figure 34: Åbn klæmmen

Figure 35: Snap fitting Figure 36: Lock Figure 37: Open the clamp

Figure 38: Klik fittingen i Figure 39: Lås Figure 40: Åbn klæmmen

Figure 41: Snap fitting Figure 42: Lock Figure 43: Open the clamp

Figure 44: Klik fittingen i Figure 45: Lås Figure 46: Åbn klæmmen

Figure 47: Snap fitting Figure 48: Lock Figure 49: Open the clamp

Figure 50: Klik fittingen i Figure 51: Lås Figure 52: Åbn klæmmen

Figure 53: Snap fitting Figure 54: Lock Figure 55: Open the clamp

Figure 56: Klik fittingen i Figure 57: Lås Figure 58: Åbn klæmmen

Figure 59: Snap fitting Figure 60: Lock Figure 61: Open the clamp

Figure 62: Klik fittingen i Figure 63: Lås Figure 64: Åbn klæmmen

Figure 65: Snap fitting Figure 66: Lock Figure 67: Open the clamp

Figure 68: Klik fittingen i Figure 69: Lås Figure 70: Åbn klæmmen

Figure 71: Snap fitting Figure 72: Lock Figure 73: Open the clamp

Figure 74: Klik fittingen i Figure 75: Lås Figure 76: Åbn klæmmen

Figure 77: Snap fitting Figure 78: Lock Figure 79: Open the clamp

Figure 80: Klik fittingen i Figure 81: Lås Figure 82: Åbn klæmmen

Figure 83: Snap fitting Figure 84: Lock Figure 85: Open the clamp

Figure 86: Klik fittingen i Figure 87: Lås Figure 88: Åbn klæmmen

Figure 89: Snap fitting Figure 90: Lock Figure 91: Open the clamp

Figure 92: Klik fittingen i Figure 93: Lås Figure 94: Åbn klæmmen

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