Mul-T-Blanket®

REF 8001-061-810

REF 8001-061-812

Operations Manual

Manual de uso

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Bedieningshandleiding

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Betjeningsmanual

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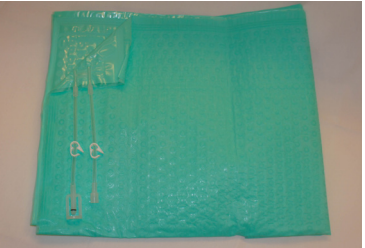
Introduction

This manual assists you with the operation of the 3T Medical Model 8001-061-810 and 8001-061-812 Mul-T-Blanket®. Read this manual before operating this product and keep a copy on file. Set methods and procedures to educate and train your staff on the safe operation of this product.

Intended use and Contraindications

Always consult the controller operations manual before using this product.

Product description

The Mul-T-Blanket is a lightweight polymer with non-woven fabric on one side. The Click-Tite® connectors provide the means to connect to the temperature controller.

Expected life

The Mul-T-Blanket has a 30 day single patient use expected life upon first use and under normal use conditions.

Specifications

Material	polypropylene, ethylene vinyl acetate, polyvinyl chloride (PVC)			
Model	8001-061-810		8001-061-812	
Patient	adult		pediatric	
Length	64 in	163 cm	33 in	84 cm
Width	25 in.	64 cm	22 in.	56 cm
Compatible controllers	Medi-Therm series temperature controller Altra® Precision Temperature Management System Note: Consult your local representative for controller availability.			
Environmental conditions	Operation		Storage and transportation	
Temperature	60 °F (16 °C) 90 °F (32 °C)		-40 °F (-40 °C) 140 °F (60 °C)	
Relative humidity	30% 75%		10% 95%	

3T Medical reserves the right to change specifications without notice.

Symbols

General warning	Caution
Refer to instruction manual/booklet	Consult instructions for use
Temperature controller	Quantity
Do not puncture	Single patient use
Batch code	Catalogue number
Date of manufacture	Manufacturer
CAUTION	
US Federal Law restricts this device to sale by or on the order of a licensed physician.	

Contact information

Contact 3T Medical Customer Service or Technical Support at +1 (734)525-9580.

3T Medical Systems, Inc.
236 Fernwood Avenue
Edison, NJ 08837
USA

To view your product manual online, see <https://3tmedical.com/technical-docs>

Have the Lot batch code (A) (Figure 1) available when you call Customer Service. Include the Lot batch code in all written communication.

Safety Precautions

WARNING

- Always check the product for leaks, tears, cuts, holes, stains, or any other damage prior to and during use.
- Always check the patient's skin condition in areas of contact with the blanket according to hospital protocol or as directed by a physician.
- Always turn or reposition the patient over the duration of therapy according to hospital protocol.
- Always keep the area between the patient and the thermal transfer device dry.
- Application of heat or cold are most effective when you apply the body wrap directly to the skin.
- Do not allow hoses to fold, kink, or to wrap around a patient.

CAUTION

- Do not use if the package is damaged.
- Always use Mul-T-Blanket products with an approved temperature controller. Using this product with other controllers has not been tested.
- Do not reuse this product on another patient to avoid the risk of cross-contamination and infection.
- Do not use this product to position or transfer a patient.
- Always pre-fill the product before applying to the patient to avoid the risk of pressure ulcers.
- Do not use three or more adult Mul-T-Blanket products at the same time to avoid the risk of water overflow when you power off the controller.
- Always consult the controller operations manual before connecting this product.
- Do not use sharp objects or pins with this product.
- Do not clean this product. If soiled, dispose of product according to hospital protocol. This is a single patient use product.
- Always use a dry absorbent sheet between the patient and the plastic side of any thermal transfer device.
- Do not block fluid pathways of the thermal transfer device with restraints.
- Always use minimal layers of sheeting. Too many layers between the patient's skin and the blanket will reduce the cooling or warming capabilities of the system.

Operations

Assessing the patient's skin condition

Note: Follow your facilities skin care protocol. The following instructions are recommendations only.

- Document initial skin assessment.
- Check for any foreign items such as medical patches, IVs, sutures.
- Check for fluids such as moisture or gels.
- Make sure that the blanket size is correct and pre-filled before applying to the patient.

Notes:

- Apply the blanket directly to the skin or add minimal layers.
- You can place the blanket over or under a patient. If you are using two blankets, you can place one over and one under.

Connecting and pre-filling the blanket (Click-Tite connectors)

- Close the pinch clamps on the blanket (Figure 2).
- Attach the connector hoses to the controller.
- Attach the connector hoses to the blanket. Insert the male coupling of blanket into the female coupling of hose with a twisting motion (Figure 3 and Figure 4).
- Snap the fitting into place (Figure 5 and Figure 6).
- With a light pull, make sure that you locked all the connectors.
- Open all the pinch clamps on the blanket (Figure 7).
- Insert the power cord into a wall outlet.
- Press the power button to turn on the controller. See the controller instructions for use.
- Check that the blanket fits with water.

Note: Iodine based disinfectants will stain the product. Staining does not affect the patient nor the use of the product.

Rechecking the patient's skin condition

Check the patient at regular intervals as directed by hospital protocol. Note any change in the skin integrity that relates to:

- Excessive moisture - dry the skin surface by wiping away the moisture
- Color of the epidermis
- Skin texture
- Patient's skin condition is acceptable to continue therapy

Check the blanket

Recheck the blanket for the following:

- position
- water status
- dry surface
- leaks
- cracks

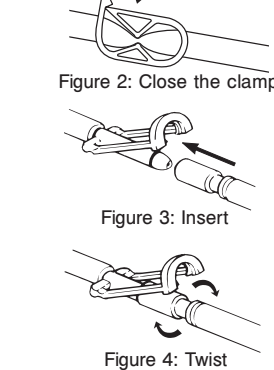
Defibrillation considerations

- Remove the blanket to expose patient's chest.
- Remove excess moisture.

Disconnecting the blanket from the hoses

- Power off the controller. Allow most of the water to drain back into the machine. (Approximately 10 minutes).
- Close blanket clamps (Figure 2).
- Close the connector hose clamps.
- Unsnap the fitting. Pull the male coupling out to disconnect (Figure 3).
- Discard blanket after use per facility protocol.

Note: Do not discard the controller connector hose after use, save for next use.

Disposing of the product

Dispose of the product in accordance with your local waste management policy.

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